



## **Life-Line Contour Shower Seats**

***Teak and Walnut Wood 18" and 26" Stainless Steel Folding Shower Seats***



### **Installation Guidelines**

- Contour seats require in-wall backing between wall studs.
- The 18" Contour seat mounts at 16" on center, so this model can be attached directly to wall studs that are 16" on center.
- The 26" Contour seat will require additional in-wall wood backing.
- The rear wooden slats on the Contour seats can be removed in order to clean the back wall when necessary.

### **Installation Instructions**

1. Pre-Installation Setup:
  - A. Place a drop cloth, plastic sheet, or other material down beneath the area where the seat will be installed.
  - B. Remove the rear slats and place the seat on the wall in the desired location. Be sure that the seat is level.
  - C. With a pencil, mark the mounting holes. Remove the seat from the wall. If drilling into tile, mark the mounting holes with a center punch.
2. Installing the Contour Seat:
  - A. Drill the mounting holes with a ceramic or masonry drill bit.

- B. Place the seat against the wall aligned with the holes and attach with the provided stainless steel mounting screws.
- C. After the seat is attached, double check that it is level. Additionally, make sure that the two mounting arms are parallel to each other and not flared in or out. Parallel mounting arms will allow the rear slats to be put back into place. (These slats can be removed later for easy cleaning of the wall).
- D. Replace the rear wooden slats.

## Safety Caution

- It is important that you give serious consideration as to the location of your Contour Shower Seat. Due to variances in local building codes and requirements, the installation instructions provided below are **ONLY** suggestions.
- We recommend seeking advice from a healthcare professional and/or a qualified contractor before beginning installation.
- This seat is intended for sitting purposes **ONLY**. ONE PERSON at a time.
- The seat is designed to withstand 500 Lbs. when properly installed. Improper installation can cause bodily harm.
- Discontinue use of the seat if visual signs of fatigue or wear become evident. Regular inspections should be made to confirm that the seat isn't pulling away from the wall. **IMMEDIATELY** discontinue use and call a qualified contractor if this starts to happen.